


## Pill box diagram

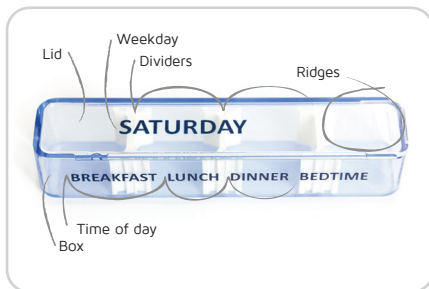


## Before use!

- Before using Medidos for the first time, we recommend that you wash the each pill box. You can wash them by hand with a mild detergent or in a dishwasher at 50 °C.
- We also recommend cleaning before refilling your pill boxes especially if your medication changes.
- Remember to keep your pill boxes away from children.
- Pay close attention to the storage requirements of your medication, some medication cannot be removed from it original packing before use. There may be temperature or photo-sensitivity requirements, if in doubt please ask your pharmacist.



- Remove the pill boxes from the cover
- Slide back lid/lids and remove completely

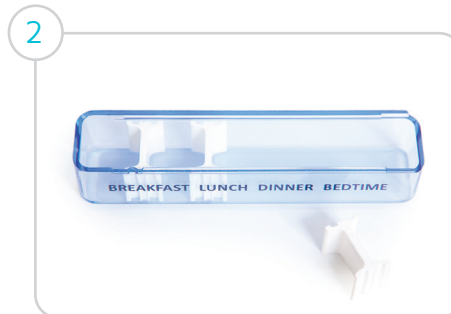

- Adjust or remove dividers as needed



- Place your medication in the appropriate compartments/boxes



- Replace the lid. The lid is secure when you feel/hear a click



- Slide back the lid so that it lines up with the appropriate divider
- Simply tip the medication into you hand



- Slide the lid to the closed position

**Tip!** You can record information about your medication on the medicine card provided